# Wireless Vibrating Personal Receiver

For use with Serene Innovations, Inc.  
**CentralAlert™** CA-360 Wireless Notification System

## Operating Guide

Model  
**CA-PX**

Thank you for purchasing this **CentralAlert™** Model CA-PX Wireless Vibrating Personal Receiver. Please read this operating guide very carefully before use and keep it for future reference. If you have any questions, please contact our Customer Care Department.

**Toll-Free Help Line:** 866.376.9271 (8AM-5PM PST, M-F)

### **Important Safety Instructions:**

When using the **CentralAlert™** Model CA-PX, to reduce the risk of fire, electric shock or injury, basic safety precautions should always be followed, including the following:

1. Follow all instructions in this operating guide.
2. **Do Not** use this device near water.
3. Clean only with a dry cloth.
4. **Do Not** block any ventilation openings. Install in accordance with the manufacturer's instructions.
5. **Do Not** install near any heat sources such as radiators, heat registers, stoves or other apparatus (including amplifiers) that produce heat.
6. Protect the power cord from being walked on or pinched. Pay particular attention at plugs, convenience receptacles and the point where they exit from the device.

## Important Safety Instructions (Cont):

7. Only use attachments/accessories specified by the manufacturer.
8. Unplug this device during lightning storms or when unused for long periods of time.
9. Refer all servicing to qualified personnel. Servicing is required when the device has been damaged such as:
  - Power-supply cord or plug is damaged.
  - Liquid has been spilled.
  - Objects have fallen onto the device.
  - The device has been exposed to rain or moisture.
  - The device does not operate normally or has been dropped.
10. **Do Not** place the unit on a surface that vibrates (e.g. dryer, washing machine, etc.). The unit should only be placed on flat non-vibrating surfaces (i.e. table, shelf, counter-tops etc.).
11. Never place this equipment in an unstable location. The equipment may fall, causing serious personal injury or death.
12. To reduce the risk of fire or electrical shock, do not expose this device to rain or moisture.
13. **Do Not** expose this device to dripping or splashing. **Do Not** place objects filled with liquids, such as vases on or near this device. Liquids can cause a failure and/or a fire hazard.

## Important Safety Instructions (Cont):


As with any electronic products, use care not to spill liquids into any part of the system.

- Do not place any open flame sources, such as lighted candles, on or near this device.

**CAUTION:** Make no modifications to the system or accessories. Unauthorized alterations may compromise safety, regulatory compliance, system performance and may void the warranty.


## Package Contents:

Your Model CA-PX Wireless Vibrating Personal Receiver is packaged with all of the items shown. If anything is missing, please contact our Customer Care Department.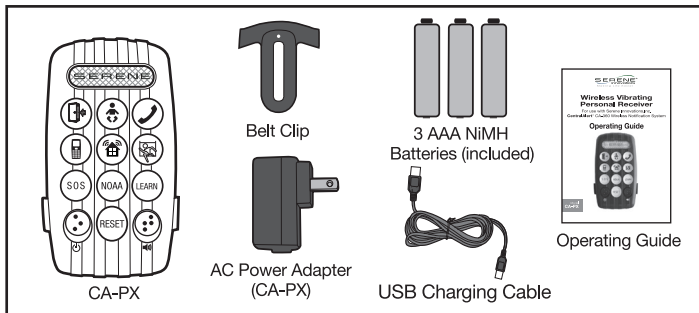


## Feature Identification:

### CA-PX Wearable Personal Receiver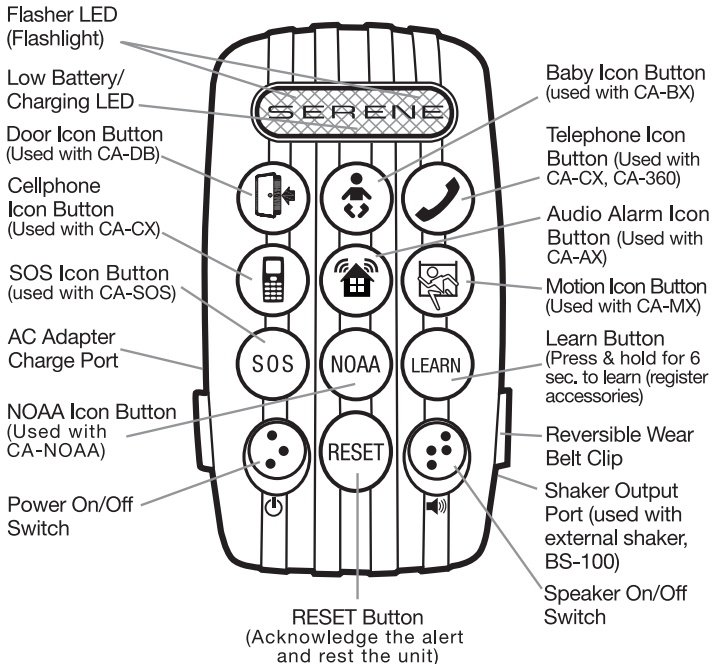

## Feature Identification (Cont):

### CA-PX Icon Buttons Braille Characters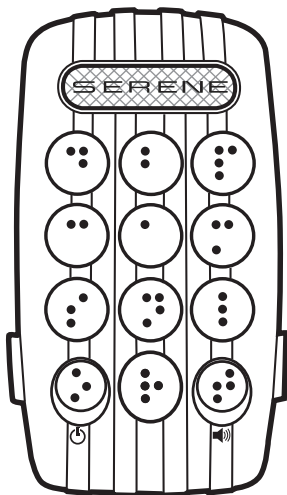

Braille Characters Embossed on Buttons

D	B	P
C	A	M
S	N	L
O	R	T


D-Door  
B-Baby  
P-Phone

C-Cell phone  
A-Audio Alarm  
M-Motion

S-SOS  
N-NOAA  
L-Learn

O-On/Off  
R-Reset  
T-Tone

## Battery Installation and Charging: CA-PX Wearable Personal Receiver

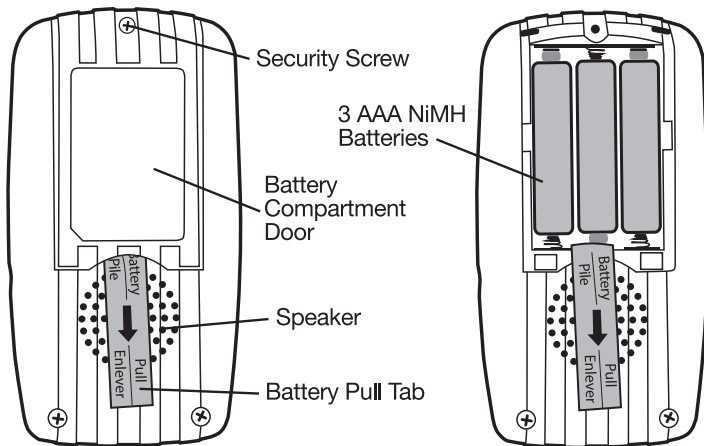Back of CA-PX

### Battery Installation:

Slide power switch to **OFF**. Pull to remove the battery tab to engage the pre-installed battery. (If the tab breaks, remove the security screw and open the battery compartment door.

## **Battery Installation and Charging (Cont):**

to remove the tab. Close the battery compartment door and reinstall the security screw).

**NOTE: Do not use regular alkaline or other non-rechargeable batteries as they will damage the unit and void its warranty. For optimal performance, batteries should be replaced once each year.**

### **Charging:**

Plug the supplied AC power adapter into an electrical outlet and connect the charging cable to the CA-PX. If this is the first time charging the CA-PX, let the batteries charge for 3-4 hours before use. Fully recharged batteries can last up to 3 – 6 days, depending upon use. We recommend re-charging the CA-PX every 3 days.


**NOTE: The CA-PX power switch must be in the ON position during charging.**

### **Low-Battery Alerts:**

The CA-PX has built-in low battery detection. The low battery indicator (which also serves as the charging indicator) will blink when the battery charge is low.

## Battery Installation and Charging (Cont):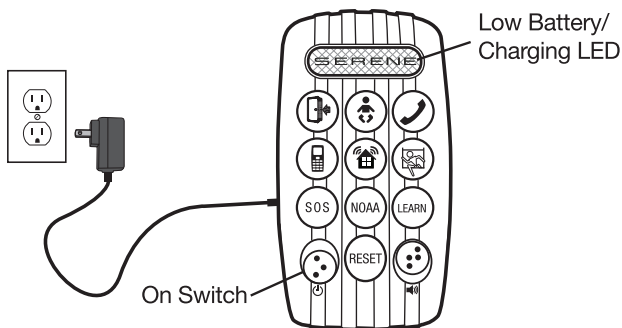

**PLEASE NOTE:** (If you add a CA-AX Audio Sensor) The CA-PX also monitors low battery warning signals transmitted from learned (or registered) CA-AX audio sensors. If a CA-AX low battery alert is detected, the CA-AX icon will blink and the user must check/replace batteries in the affected unit.

## Connection And Setup:

### CA-PX Applications:

The CA-PX provides four (4) types of alerting outputs: speaker, flasher, blinking icons and vibrations. The CA-PX is ready to use right out of the box. New sensor accessories must be learned (or registered) to CA-PX before they can be

## **Connection And Setup (Cont):**

recognized. When using the CA-PX as a stand alone receiver (without the CA-360 master unit), it can support as many accessory units as the CA-360.

### **Learning Accessories:**


#### **CA-PX as a Stand-Alone Receiver**

##### **Example Application:**

1. This application demonstrates how a simple wireless notification system can be created using just one CA-PX as a receiver unit. It notifies you if an audio alarm is activated.
2. The example (A) shows a simple system comprised of a personal receiver (CA-PX), and an audio sensor (CA-AX).
3. To further enhance the system effectiveness and user safety, both registered and unregistered CA-AX audio sensor wireless signals can be detected by the CA-PX.

Note 1: Model CA-AX is an optional accessory and is sold separately.

## Connection And Setup (Cont):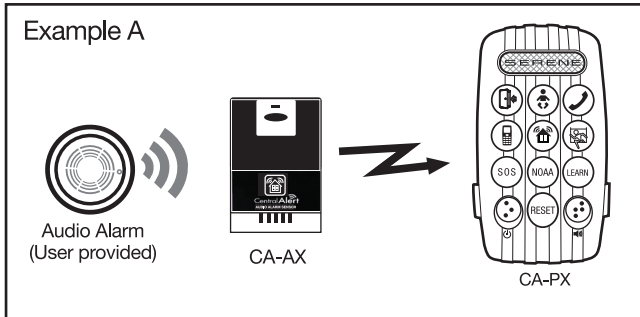

Follow the steps below:

1. Install batteries in CA-AX
2. Power On the CA-PX.
3. Press and hold the Learn button for 6 seconds until it vibrates (and beeps if the speaker is On) to enter the Learn mode. The Learn button LED will blink for about 20 seconds, after which it will exit the Learn mode.
4. Press and hold the test button on the accessory CA-AX (audio sensor) for more than 10 seconds until CA-PX flashes, vibrates, and beeps (if speaker is On) twice. This confirms the learning.

## Connection And Setup (Cont):

5. Press the Reset button on the CA-PX to exit the learning mode at any time.
6. Test the system - Press the Test button on CA-CX (for this example). Verify that the corresponding icon button on the CA-PX is activated. The CA-PX will flash, vibrate and beep (if speaker is On) for about 45 seconds. Press **RESET** to stop at any time.
7. Repeat the steps above to learn (or register) and test more accessories as desired.

## Using CA-PX as an accessory to the CentralAlert™ Tabletop Notification System:


The CA-PX can also be used as an accessory to the CA-360 **CentralAlert™** Tabletop Notification System. In this configuration, the CA-360 becomes the Master Unit and the CA-PX becomes a remote receiver and part of the system that receives alerts just as the CA-360

### Example Application:

Each CA-PX must be learned (or registered) to the CA-360 system before before it can be recognized (Up to 10 CA-PX may be added to one CA-360 master unit).

## Connection And Setup (Cont):

**Note: All previously learned accessories in the CA-PX will be lost during this process, and all lost accessories must be learned again through the CA-360 master unit.**

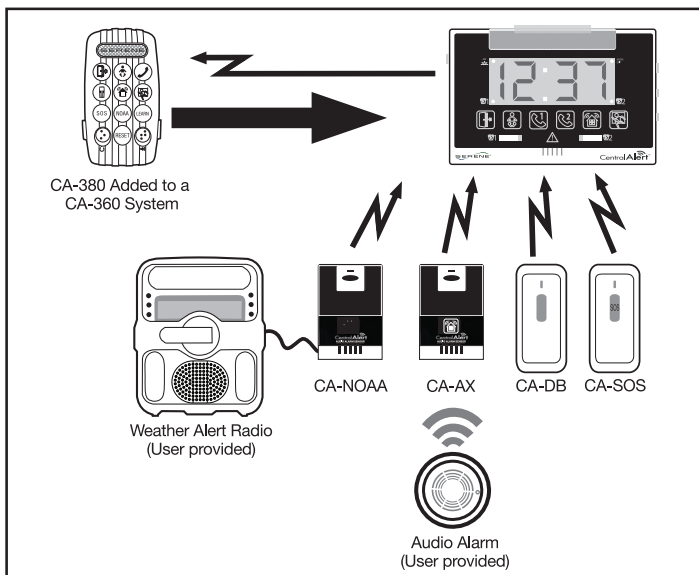## Connection And Setup (Cont):

### Follow the steps below:

1. Place CA-360 master unit in the clock mode (**RUN** position), Press and Hold the **SOS / TEST** button for 6 seconds until it beeps. It will display “**LEAR**” and ready to learn accessories.
2. Power On the CA-PX.
3. Press and hold the **Learn** button on the CA-PX for at least 10 seconds until CA-360 beeps twice, and the display shows “**SUB**” - confirming registration is successful.
4. Press **RESET** button on both CA-360 unit and the CA-PX to end the Learn process.
5. Testing the system: Press and Hold the **SOS** button on the CA-PX for 2 seconds. This should activate an alert on both the CA-PX and CA-360 units. Press the **RESET** button on either unit to stop and cancel the alert.
6. Press an accessory (such as a doorbell) that was previously registered to the CA-360 master unit. Verify that both CA-360 and the CA-PX alerts are activated for 45 seconds, or until the **RESET** button is pressed.
7. The CA-PX is now ready and will alert the user whenever a wireless signal is detected.

**Reminder:** All previously learned accessories in the CA-PX will be lost during the process of adding the

## Connection And Setup (Cont):

CA-PX into a CA-360 alert system. All lost CA-PX accessories will need to be learned again into the system through the CA-360 master unit.

### Testing

1. While in standby mode, press and hold the **SOS** icon button for 2 seconds. If the CA-PX was registered to a CA-360 system, all receiver units registered in the system are activated for 45 seconds.
2. Press the **RESET** button to stop and exit the alert.

### Operations:

Power **On/Off** and Speaker **On/Off** switches

Switch	Actions	CA-PX Outputs	Weak Battery Condition
Power	OFF  ON 	Turning On-Flash once, Vibrate once, and Beep once	Vibrate Once
Speaker	OFF  ON 	Turning On-Beep once Turning Off-Vibrate once	Vibrate Once

## Turn Flashlight On/Off :

The flasher on the top of the unit can be used as a flashlight.

1. Press **RESET** button once to turn on.
2. Press the **RESET** button once again to turn off.

## Sending SOS signal (CA-PX in Standby Mode):

To page other receivers for help, press and hold the **SOS** button for 2 seconds. A wireless **SOS** signal will be transmitted to nearby registered receivers (CA-PX, CA-RX, CA-360).

## Optional Bed Shaker Operation:

With an optional bed shaker (BS-100) attached, CA-PX can activate the bed shaker while charging. This provides total notification, even while you sleep.

## Alert Signals Acknowledge:

Upon receiving a wireless notification signal from any accessory:

1. The unit activates a unique motor vibration, blinking icon button, flasher and sound (if speaker is On).
2. After reviewing the blinking icon button, the user can then press the Reset button to acknowledge and stop all alert outputs. The blinking icon button continues to stay lit for 5 more seconds.

## Alert Signals Acknowledge (Cont):

3. For a Deaf/Blind user who cannot visualize the blinking icon button, the user will need to press each icon button sequentially to determine the activated icon button. When the correct icon button is pressed, the vibration will become continuous – indicating this is the correct one. The user can then press the Reset button to acknowledge, and stop all alert outputs. (**Note:** Each button is embossed with a Braille character).

## Recall Last Activated Alert :

The last activated alert may be recalled for review purposes.

1. Press and hold the **RESET** button for 2 seconds and it will recall the last activated alert. Unique vibration, blinking icon button and speaker sound will be repeated for 45 seconds. (**Note:** The flasher will not flash during recall).
2. A Deaf / Blind user can press each icon button sequentially to determine which icon button is active. When the user presses the correct icon button, the vibration will become continuous – indicating this is the correct one. Each button is embossed with Braille.
3. Press the **RESET** button to stop and exit the mode.

## Vibration Pattern Training:

The CA-PX offers an exclusive training feature that allows the users to familiarize themselves with the vibration pattern of each alert or icon button.

Icon	Alerting Accessory	Vibrating Patterns
	CA-AX	^^^^^^ (continuous)
SOS	CA-SOS	^^^^--- pause, repeats
	CA-MX	^^^^ pause, repeats
NOAA	CA-NOAA	^--- pause, repeats
	CA-BX	^^--- pause, repeats
	CA-DB, CA-DX CA-DXH	^^ pause, repeats
	CA-CX, CA-360	--- pause, repeats
	CA-CX	--- --- pause, repeats
^ = 0.75 seconds short vibration    pause = 1 second rest -- = 2 second vibration		

1. Press and hold the **LEARN** button while unit is Powered On. The **LEARN** button LED will be On.

## Vibration Pattern Training (Cont):

2. Press and release any of the alert icon buttons. This activates the vibration pattern for 45 seconds, so the user can feel and learn. (**Note:** Button icon and speaker sound will also be active.)
3. Press the **RESET** button to stop or press another button to learn a new vibration pattern.
4. To exit the training session, Power Off the CA-PX unit.
5. CA-PX will not respond to any wireless alert signal during the training session.

## Low-Battery Alerts :

The CA-PX unit has a built-in, low battery detection to monitor its own battery condition. The low battery indicator (also serves as the charging indicator) will blink if the CA-PX battery is running low.

The CA-PX also monitors low battery warning signals transmitted from learned (or registered) CA-AX audio sensors. If a CA-AX low battery alert is detected, user must then check all the batteries in the CA-AX accessories used in the system to eliminate this alert.

## Low-Battery Alerts (Cont):

Event	Indicator	Visual	Notifications
CA-PX Low Battery		Continuous Blinking	Flashes, Vibrates & Beeps 4 times (if speaker is on). Then repeats every 15 minutes (No reset allowed).
Registered CA-AX Low Battery Signal		Continuous Blinking	Flashes, Vibrates & Beeps 4 times (if speaker is on) with 20 second pause in between, then repeats every 15 minutes.
To Temporary Cancel CA-AX Low Battery Alert	Hold button more than 5 seconds to suspend this alert until the next signal is received again.	Stop Blinking	Flashes, Vibrates & Beeps once (if speaker is on).

## **Clear Internal Memory** (Factory Default)

If the CA-PX cannot learn (or register) any accessories or register into a CA-360 system after several attempts, the internal memory of the CA-PX may need to be cleared.

Press and release the Learn button on the CA-PX for at least 18 times in quick succession until it flashes, vibrates and sounds a beep (if speaker is On). Then relearn all accessories.

## **Trouble Shooting FAQ :**

### **What type of batteries do I need for the CA-PX?**

**Ans:** Install 3 AAA-rechargeable batteries (NiMH). For optimal performance, it is recommended that the batteries be replaced once a year.

### **How many accessories can the CA-PX support?**

**Ans:** The CA-PX can support up to 10 Remote-Receivers (CA-RX), 10 Personal pagers (CA-PX), 10 audio sensors (CA-AX), CA-AXv2), 4 SOS pendants (CA-SOS), 3 Baby-Sound sensors (CA-BX), 1 landline/cellphone sensor (CA-CX), and 1 NOAA sensor (CA-NOAA).

### **I just bought an extra doorbell or door knock sensor, but it did not work. What do I need to do?**

**Ans:** All new accessory devices must be learned or registered to the CA-PX unit (and CA-360 if used as the master unit)

## **Trouble Shooting FAQ (Cont):**

before they can be recognized or work. See the section in this manual for instructions as how to learn or register an accessory device.

### **How do I clear all registered accessories from the CA-PX?**

**Ans:** Press and release the Learn button at least 18 times in quick succession until it flashes, vibrates and sounds once (if Speaker is On).

### **Does CA-PX work with a CentralAlert™ CA-360 system?**

**Ans:** Yes, the CA-PX can be used as a remote receiver to a **CentralAlert™** CA-360 master unit. Follow the steps described in the CA-PX Application section to complete the process.

### **How far can the CA-PX be located from CA-360?**

**Ans:** About 150 feet in open space (metal inside a wall can reduce this distance).

### **What is the maximum distance between accessories and the CA-PX?**

**Ans:** About 150 feet in open space (metal inside a wall can reduce this distance).

## Trouble Shooting FAQ (Cont):

### Can I place CA-PX next to my Wi-Fi router?

**Ans:** No. Please keep all other electronic equipment at least 5 feet away.

### Does CA-PX offer a wireless signal repeater function?

**Ans:** Yes. CA-PX, CA-RX and CA-360 units offer a wireless repeater/relay function whereby each unit can re-broadcast the received wireless signals to other similar receivers. This repeater function expands the coverage area of the entire system.

## Cleaning and Maintenance:

- The surface of the device can be cleaned with a soft, dry cloth.
- **Do Not** use any sprays near the device. **Do Not** use any solvents, chemicals, or cleaning solutions containing alcohol, ammonia or abrasives.
- **Do Not** allow liquids to spill into any openings.

## Other Accessories for the Model CA-PX :

<b>MODEL</b>	<b>DESCRIPTION</b>
CA-AX	Audio Alarm Sensor
CA-BX	Baby Sounds Sensor
CA-CX	Landline/Cell Phone Ringer/Flasher Sensor Transmitter
CA-DB	Weather Resistant, Wireless Doorbell
CA-DX	Door Knocker Sensor
CA-DXH	Door Hanging Door Knocker
CA-MX	Motion Detector
CA-NOAA	Emergency Weather Warning Sensor
CA-RX	Portable Remote Receiver
CA-SOS	Emergency Help Button Sensor
BS-100	Bed Shaker

## Specifications :

Technology.....	2.4 GHz Smart Code
Bed Shaker Output.....	5 VDC, 150mA
Battery Backup.....	3 x AAA Size NiMH rechargeable
Battery Use Time.....	3 to 6 days depending on usage
AC Adapter Input.....	100-240 VAC, 50 / 60 Hz
AC Adapter Output.....	5VDC 600-1000 mA
Operating temperature.....	+10 to 40 C
Dimensions.....	Approximately 57 x 108 x 32 (W x D x H mm) w/belt clip
Weight.....	Approximately 110 grams (with battery)

Specifications are subject to change.

## Warranty Service Information:

Your **CA-PX Wireless Vibrating Personal Receiver**. comes with a one year limited warranty. We warrant during this warranty period from the date of purchase to the original consumer that these products will be free from defects in materials and workmanship under intended home use. In the event that these products fail to function properly within one year of original purchase due to defects in materials or workmanship, return the unit (freight prepaid) with proof of purchase (sales receipt or packing slip, no exceptions) to Serene Innovations, Inc.. Serene Innovations, Inc. will either repair or

## Warranty Service Information (Cont.):

replace the unit (with a refurbished unit or unit of equal condition) and return it to you (using UPS/USPS ground shipping) at no cost to you, if the unit is returned within 30 days of purchase. If the unit is returned after 30 days of purchase but within the warranty period, there will be a warranty handling charge of U.S. \$9.95 for each return.

This warranty does not apply to any product that has been accidentally damaged due to: abuse, misuse, negligence, use on electrical frequency or voltage other than marked on product and/or described in this manual. Defects or errors caused by unauthorized alterations, repairs and/or tampering are also not covered by this warranty.

### **Incidental or Consequential Damages:**

Neither Serene Innovations, Inc. nor the retailer, dealer or selling distributor has the responsibility for any incidental or consequential damages, including, without limitation, commercial loss of profit or for any incidental expenses, loss of time or inconvenience. Some states do not allow exclusion or limitation of incidental or consequential damage, so the above limitation or exclusion may not apply to you.

This warranty gives you specific legal rights. You may have other legal rights that vary from state to state. If a defect covered by this warranty should occur, promptly contact a Customer Care Representative by phone or log on to [www.sereneinnovations.com](http://www.sereneinnovations.com) to obtain a Return Merchandise Authorization number and shipping instructions before re-turning the product to us. Any shipment without a RMA will not be accepted and will be returned to you at your expense.

## Warranty Service Information (Cont.):

Any authorized returned product must be accompanied with proof of purchase and a brief description of the problem. For out of warranty repair and service, please contact our Customer Care Department for directions.

## FCC Information for USA Customers Only:

### Part 15 of FCC Rules Information

This device complies with Part 15 of the FCC rules. Operation is subject to the following two conditions: (1) this device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation. This equipment has been tested and found to comply with the limits of a Class B digital device, pursuant to Part 15 of FCC rules. These limits are designed to provide reasonable protection against harmful interference in residential installation.

This equipment generates, uses, and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, you may try one of the following methods to correct the interference:

1. Where it can be done safely, re-orient the receiving television or radio antenna.
2. To the extent possible, relocate the television, radio, or other receiver, with respect to the cordless phone.
3. Connect the cordless phone AC adapter into a different electrical outlet.
4. Consult the dealer or an experienced radio/TV technician for help. **Modification:** Any modification not expressly approved by the manufacturer of this device could void the user's authority to operate the device.

### **NOTICE: The Industry Canada label identifies certified equipment.**

This certification means that the equipment meets telecommunications network protective, operational, and safety requirements as prescribed in the appropriate Terminal Equipment Technical Requirements document(s). The Department does not guarantee the equipment will operate to the user's satisfaction.

## FCC Information for USA Customers Only (Cont.):

Before installing this equipment, users should ensure that it is permissible to be connected to the facilities of the local telecommunications company. The equipment must also be installed using an acceptable method of connection. The customer should be aware that compliance with the above conditions may not prevent degradation of service in some situations. Repairs to certified equipment should be coordinated by a representative designated by the supplier. Any repairs or alterations made by the user to this equipment, or equipment malfunctions, may give the telecommunications company cause to request that the user disconnect the equipment. Users should ensure, for their own protection, that the electrical ground connections of the power utility, telephone lines, and internal metallic water pipe system, if present, are connected together. This precaution may be particularly important in rural areas.

**Caution:** Users should not attempt to make such connections themselves, but should contact the appropriate electric inspections authority, or electrician, as appropriate. The REN for this amplified telephone is stated on the IC regulatory label located on the bottom of the product.

**NOTICE:** The Ringer Equivalence Number (REN) assigned to each terminal device provides an indication of the maximum number of terminals allowed to be connected to a telephone interface. The termination on an interface may consist of any combination of devices, subject only to the requirement that the sum of the Ringer Equivalence Numbers of all the devices does not exceed 5.

This Class B digital device complies with Canadian ICES-003.

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